

## Frequently Asked Questions



1. AMS 700™ with MS Pump Series Inflatable Penile Prosthesis Patient Manual.  
American Medical Systems, Inc. 2019.

All cited trademarks are the property of their respective owners.

CAUTION: The law restricts these devices to sale by or on the order of a physician. Indications, contraindications, warnings and instructions for use can be found in the product labelling supplied with each device. Products shown for INFORMATION purposes only and may not be approved or for sale in certain countries. This material not intended for use in France.

Product available in the European Economic Area (EEA) only.

Please check availability with your local sales representative or customer service.

All images are the property of Boston Scientific. All trademarks are the property of their respective owners.

MH-1086002-AA. Produced by Gosling.

**Boston  
Scientific**  
Advancing science for life™

[www.bostonscientific.eu](http://www.bostonscientific.eu)

2022 Copyright © Boston Scientific  
Corporation or its affiliates.  
All rights reserved.  
DINURO2484EA

**THE AMS 700™ SERIES PENILE IMPLANTS  
FOR TREATMENT OF ERECTILE DYSFUNCTION**

# Frequently Asked Questions (FAQ) About Your Procedure

## Is this an inpatient procedure?

Usually it is; however, this is a decision that will be made between you and your urologist.

## How long does it take to perform a penile implant procedure?

Implantation of a penile prosthesis involves a surgical procedure typically lasting anywhere from 30 minutes to two hours.

## Will anyone be able to visually tell I have an implant?

Once in place, your implant will be completely undetectable. It's fully concealed in the body. No one will know unless you tell them – even in the changing room.

## How much pain will I experience?

You may experience pain or discomfort at the operative site(s) during the early time after surgery and when you first use your prosthesis. In most cases, the pain goes away within a few weeks of surgery; however, some cases of chronic pain have been reported.<sup>1</sup>

## How long after the surgery must I wait before I use the implant?

Your doctor will discuss when you will be able to use your device. With a penile prosthesis, most doctors will recommend that you wait four to six weeks before having intercourse.<sup>1</sup>

## Is the inflation process painful with AMS 700?

To inflate an AMS implant, you squeeze the pump in your scrotum, which will send fluid into the cylinders in the penis. This requires good manual dexterity but should not be painful. If you experience pain, contact your doctor.<sup>1</sup>

## Are Boston Scientific penile implants MRI compatible?

Boston Scientific implants are MRI conditional – meaning a patient with this device can be scanned under certain conditions but not all. Please consult with your urologist.

## Common Questions To Ask Your Physician

### What do I need to do before my procedure?

### What are the risks of the surgery?

### How satisfied are patients with the results of this procedure?

### What are some possible side effects?


### Will I be able to drive myself home after the procedure?

### What do I need to do to ensure proper healing?

## Additional Resources

### Living with a Device

If you would like to learn more or have questions related to what it is like to live with an implant, visit [www.EDTreatments.com](http://www.EDTreatments.com).

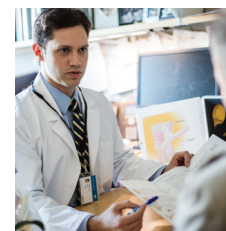

Please contact your physician at any time prior to or after your procedure with any questions.